

haakaa®

# PREMIUM SILICONE PUMP AND BOTTLE PACK

## Product Manual English

<b>Product Name:</b>	Premium Silicone Pump and Bottle Pack
<b>Materials:</b>	Bottle Body - Medical-Grade Silicone, PPSU Pump Flange - Medical-Grade Silicone Anti-Colic Nipple - Medical-Grade Silicone Sealing Disk - Medical-Grade Silicone Attachment Ring - PP Feeding Spoon - Medical-Grade Silicone Sippy Spout - Food-Grade Silicone, Stainless Steel Flower Stopper - Food-Grade Silicone Replacement Cap - PP
<b>Volume:</b>	Silicone Baby Bottle - 5.4 oz./160ml Silicone Breast Pump - 8 oz./250ml
<b>Dimensions:</b>	Silicone Baby Bottle - 83 × 79 × 128mm Silicone Breast Pump - 84 × 84 × 210mm
<b>Colours:</b>	Peach/Grey

### Product Description:

The Haakaa Premium Silicone Pump and Bottle Pack combines all of your favourite Gen. 3 bottles and attachments into one handy pack! Our Gen. 3 Pump and Bottle Pack is the world's first 5-in-1 feeding system. With a quick twist of the top, you can swap the top of your bottle for any of the Gen. 3 range of optional interchangeable attachments! Choose from our famous breast pump flange that lets you express directly into the bottle, a baby bottle nipple to feed your little one expressed milk, a feeding spoon dispenser for when your baby moves on to solids, a sippy bottle top to help your independent toddler transition from a bottle to a cup or sealing disks to preserve any leftover food or milk after feeding!

### Features:

- Made with premium medical-grade silicone.
- Multifunctional 5-in-1 feeding system.
- Easy and effective milk expression and feeding.
- Compatible with all Gen. 3 Silicone Bottle Range attachments.
- Compact, portable and convenient.
- BPA, PVC and phthalate-free.

### Cleaning and Care:

Wash before and after each use. Although dishwasher safe, we recommend hand washing in warm, soapy water and rinsing and drying thoroughly. Do not use any bleach-based cleaners or tablets to sterilise or clean your Haakaa products. To sterilise parts (NOT automatic straw), use a steam steriliser or boil in water for 2-3 minutes. To sterilise automatic straw, boil in water for 2-3 minutes. Take care handling automatic straw after sterilising as stainless steel can reach high temperatures - DO NOT microwave sterilise automatic straw.

### Caution:

- ⊙ Check product condition regularly. Replace if this product shows any signs of damage.
- ⊙ Do not store near any sharp objects.
- ⊙ Only soft bristle brushes or soft sponges should be used to clean this product. Hard scourers may scratch the surface.
- ⊙ This product is not a toy. Do not use this product for anything other than its intended use.
- ⊙ Store product in a cool dry place and avoid direct sunlight.
- ⊙ Adult supervision is required.
- ⊙ This product contains stainless steel parts, please do not microwave.

**NOTE:** As silicone can stick to itself, the nipple hole or valve opening may seal. To fix, gently rub the end of the nipple or valve between your fingers until the hole reopens. We recommend replacing the nipple every 3 months.

Volume measurements are an approximation and should be used for general reference only. Always follow your country's guidelines and recommendations to safely express and store breastmilk.